

## Main shortcuts

Want the shortcuts to work? Make sure to only use alphanumeric keys, not the numeric pad.

### General

Ctrl+Z	Undo
Ctrl+Y	Redo

### Display - Show/hide elements in the 3D view

C	Upper model & restoration
V	Lower model & restoration
B	Bite
I	Contacts
N	Full restoration
Alt+C	Upper model
Alt+V	Lower model
Space bar	Selected restoration

### Design - Enables toolkits during design

Ctrl+1	Fit
Ctrl+2	Shape
Ctrl+3	Sculpt
Ctrl+4	Occlusal
Ctrl+5	Proximal
Ctrl+6	Clone wax-up
Ctrl+7	Duplicate
Ctrl+9	Abutment

### Fit

Left-click	Translate
Ctrl+left-click	Rotate
Shift+left-click	Scale

### Shape

Shift+handle	Select same-color handles
Ctrl+handle	Select same-meridian handles
Alt+handle	Relocate

### Sculpt

Ctrl+left-click	Remove material
Shift+left-click	Smooth

### Margin line

Right arrow	Go to next handle
Left arrow	Go to previous handle
Up arrow	Move handle up
Down arrow	Move handle down
Delete	Delete handle

### Scope and intensity tools

Left-click+scroll up	Increase scope
Left-click+scroll down	Decrease scope
Right-click+scroll up	Increase intensity
Right-click+scroll down	Decrease intensity

### Scan - Enable toolkits during scanning

Ctrl+1	Trim
Ctrl+2	Sculpt
Ctrl+3	Close holes
Ctrl+4	Erase
Ctrl+7	Duplicate

### Cut plane panel

G	Grid
Ctrl+P	Alternate camera
F5	Clip plane 1
F6	Clip plane 2
F7	Clip plane 3
F8	Clip plane 4