

Main shortcuts

Want the shortcuts to work? Make sure to only use alphanumeric keys, not the numeric pad.

General

Ctrl+Z	Undo
Ctrl+Y	Redo

Display - Show/hide elements in the 3D view

C	Upper scan & restoration
V	Lower scan & restoration
B	Bite
I	Contacts
N	Full restoration
Alt+C	Upper scan
Alt+V	Lower scan
Space bar	Selected restoration

Design - Enables toolkits during design

Ctrl+1	Fit
Ctrl+2	Shape
Ctrl+3	Sculpt
Ctrl+4	Occlusal
Ctrl+5	Proximal
Ctrl+6	Clone wax-up
Ctrl+7	Duplicate
Ctrl+9	Abutment

Fit

Left-click	Translate
Ctrl+left-click	Rotate
Shift+left-click	Scale

Shape

Shift+handle	Select same-color handles
Ctrl+handle	Select same-meridian handles
Alt+handle	Relocate

Sculpt

Ctrl+left-click	Remove material
Shift+left-click	Smooth

Margin line

Right arrow	Go to next handle
Left arrow	Go to previous handle
Up arrow	Move handle up
Down arrow	Move handle down
Delete	Delete handle

Scope and intensity tools

Left-click+scroll up	Increase scope
Left-click+scroll down	Decrease scope
Right-click+scroll up	Increase intensity
Right-click+scroll down	Decrease intensity

Straumann Nova™

Scan - Enable toolkits during scanning

Ctrl+1	Trim
Ctrl+2	Sculpt
Ctrl+3	Close holes
Ctrl+4	Erase
Ctrl+7	Duplicate

Cut plane panel

G	Grid
Ctrl+P	Alternate camera
F5	Clip plane 1
F6	Clip plane 2
F7	Clip plane 3
F8	Clip plane 4